

# Lateral Extensions Fitted to Correct Fetlock Varus Deviation



1. 7 week old foal fetlock varus. Lateral extensions required L and R.



2. RF fetlock varus. hoof trimmed.



3. RF lateral wall keyed for strong attachment.



4. Imprint Adhesive applied to extension area including lateral sole.



5. Imprint Hoof Repair plastic moulded to create custom extension. Freeze hardened then allowed to weight bear.



6. Solar aspect of lateral extension. Concavity is maintained. Screw is for grip.



7. Imprint Extension beginning to bring limb into alignment.



8. Lateral extensions fitted.



9. Posterior view, extensions newly fitted.



10. Extensions at 3 weeks.



11. Re-trimmed and new extensions fitted at 3 weeks.

Imprint Equine Foot Care  
Town Forge, High Street  
Malmesbury, Wiltshire, SN16 9AT

Tel: 01666 822953

[www.facebook.com/ImprintEquineFootCare](http://www.facebook.com/ImprintEquineFootCare)  
[www.imprintshoes.co.uk](http://www.imprintshoes.co.uk)

**IMPRINT**<sup>®</sup>  
+  
**EQUINE FOOT CARE**